

Install and Use Switch Interface Pro 6.0

What is Switch Interface Pro?

The Switch Interface Pro 6.0 allows specially designed switch activated software to use customized settings in place of keyboard commands.

Product Description

- Switch Interface Pro 6.0 supports up to five switches with standard 1/8" (3.5 mm) plugs to fit the five jacks in the interface.
- The push-button on the back of the unit controls the functionality.
- Separate LED lights mounted on the unit show you the active setting.
- Packaging provides a standard USB connector with a three-foot cable.
- Includes Dual Lock heavy-duty, velcro®-type fastener to secure the Switch Interface Pro to table or computer.

Installation

1. Locate the USB port on your computer.
2. Plug the end of the Switch Interface Pro 6.0 USB cable into the USB port on your computer.
3. Complete the installation by following the instruction below for your OS:


(Windows) If your computer asks you to add new hardware through the Add New Hardware Wizard, follow the steps in the Wizard to continue.

(Macintosh) Close out of the Keyboard Setup Assistant by clicking the red radio button.

Note: After attaching your Switch Interface Pro 6.0, the USB drivers will automatically be updated to allow your new Switch Interface Pro 6.0 to work. You will see the lights on the unit flash while your computer is updating its drivers. Once the lights stop flashing, your interface is ready to use.

Note: If the lights continue to flash indefinitely, your computer experienced a problem with one of the updates. If this happens, reboot the computer to try again.

Switch Jacks

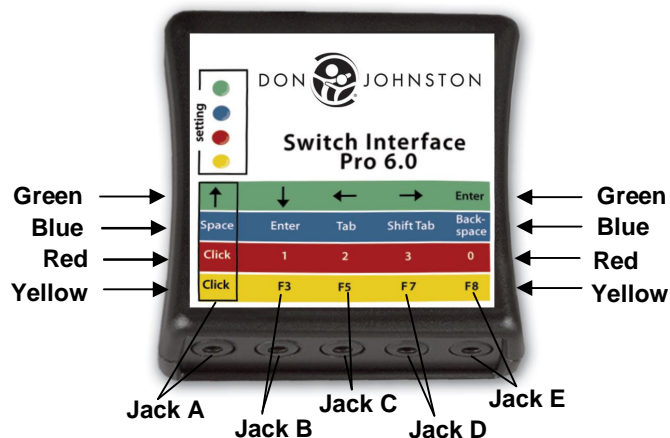

How To Use

To use with any Don Johnston Incorporated switch-enabled software, simply plug your switch into jack A in the default yellow setting, launch the program and begin scanning!

Switch Interface Pro 6.0 also works with various software titles. Use the following tables as a guide. Titles are available for both Windows and Macintosh unless otherwise noted.

If your software requires you to change the color setting (green, blue or red) as indicated in the table, do this by pressing the push-button on the back of the unit. When you change the color setting, refer to the active function line by color to determine the current function of each switch.

Use One Switch

Software	Program Settings	Color Settings	Jack A	Jack B	Jack C	Jack D	Jack E
All My Words (Crick)	Scanning ON	Blue		•			
ATI Assessment (Stage 1)	Scanning ON	Red	•				
ATI Assessment (Stage 1 Update)	Scanning ON	Red		•			
ATI Assessment (Stages 2-7)	Scanning ON	Red	•				
BuildAbility [®] Read Published Stories	N/A	Yellow or Red	•				
Camp Frog Hollow [®]	Scanning ON	Yellow or Red	•				
Circletime Tales Deluxe	Scanning ON	Red	•				
Clicker 5	Single Switch Scan	Yellow				•	
Judy Lynn Software	Scanning ON	Yellow				•	
KidTech Titles	Scanning ON	Red		•			
Laureate	Scanning ON	Red	•				
Press-to-Play Series	ANY	Yellow or Red	•				
R.J. Cooper & Assoc.	Scanning ON	Yellow or Red	•				
Simon S.I.O. [®]	Scanning ON	Yellow or Red	•				
SimTech Single Switch Software	Scanning ON	Yellow or Red	•				
SoftTouch Titles	Switch Mode	Red		•			

Software	Program Settings	Color Settings	Jack A	Jack B	Jack C	Jack D	Jack E
Boardmaker Speaking Dynamically Pro	Scanning ON	Yellow	●				
Start-to-Finish Series (Gold & Blue) [®]	Scanning ON	Yellow or Red	●				
UKanDu [®] Little Books	Scanning ON	Yellow or Red	●				
UKanDu [®] Switches, Too! Series	Scanning ON	Yellow or Red	●				

Use Multiple Switches

Software	Program Settings	Color Settings	Jack A	Jack B	Jack C	Jack D	Jack E		
ATI Assessment (Stages 5 & 7)	Scanning OFF	Blue		Select	Advance				
Camp Frog Hollow [®] (At Story Screen)	Scanning OFF	Red		Reread page	Page forward	Page back	Quit		
Clicker 5	Scanning OFF	Yellow				Advance	Select		
Simon S.I.O. [®]	Scanning OFF	Green					Select		
Start-to-Finish [®] Series (Gold & Blue)	Scanning OFF	Red					Quit		
UKanDu [®] Little Books	Scanning OFF	Red	At Main Menu, selects...						
				1 st story	2 nd story	3 rd story			
			At Story screen, selects...						
				Left choice	Right choice			Return to menu	
			At end of story, selects...						
				Reread story	Print story	Return to menu	Go to Quit screen		
UKanDu [®] Switches, Too! Series	Scanning OFF	Red	At Story Screen	At Main Menu, selects...					
			Advances Action	1 st story	2 nd story	3 rd story	Quit		

Note: When using Speaking Dynamically Pro or Writing With Symbols (Mayer-Johnson), please refer to software documentation for instructions for setting up multiple switches.

FCC STATEMENT

DECLARATION OF CONFORMITY WITH FCC RULES FOR ELECTROMAGNETIC COMPATIBILITY

The product, Switch Interface Pro 6.0, to which this declaration relates:
Complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

CE DECLARATION OF CONFORMITY

The product, Switch Interface Pro 6.0, is in conformity with Generic Emissions Standard EN 50081-1 and with Generic Immunity Standard EN55024.

Warranty

The SWITCH INTERFACE PRO 6.0 is under warranty for one (1) year from the date of purchase. This warranty is valid against any manufacturer defects and malfunctions not the result of abuse or mishandling. A service fee is charged for repairs out of warranty.

All claims must provide proof of purchase, including vendor order number and purchase date. A return authorization number is required and can be obtained by calling the Customer Success department at Don Johnston Incorporated.

Except for the above, Don Johnston makes no warranties, express or implied, including warranties as to the merchantability or as to the fitness of the products for any particular use or purpose, and shall not be liable for any loss or damage, directly or indirectly, from the use of such products or for consequential damage.

Inspection Period

You have thirty (30) days from date of purchase to examine the SWITCH INTERFACE PRO 6.0. If this product does not meet your needs, call Don Johnston Customer Success at 800-999-4660 for return authorization within the inspection time. We reserve the right to refuse any shipment that does not have return authorization. In order to receive full credit, all components must be returned in new condition within the inspection period. Orders returned past the inspection period will not be accepted for full credit.

For Your Records	1002
Date Received: _____	
Purchased by: _____	
P.O. Number: _____	
Customer Number: _____	
Order Number: _____	

Test Results

The Switch Interface Pro 6.0 does comply with the radiated emissions requirements of EN 50081-1, the radiated immunity and the electrostatic discharge requirements of EN 55024 and the radiated RF emissions requirements of EN 55022 for Class B equipment over the frequency range from 30MHz to 1GHz.

Trademarks

The Leader in Learning Intervention Resources, Forgetful & Friends, Simon S.I.O., Circletime Tales Deluxe, Humpty Dumpty & Friends and Press-to-Play are trademarks of Don Johnston Incorporated. The Don Johnston logo, Eensy & Friends, Camp Frog Hollow, BuildAbility, Start-To-Finish and UKanDu are registered trademarks of Don Johnston Incorporated.

Macintosh is a registered trademark of Apple Computer, Inc.

Windows is a registered trademark of Microsoft Corporation.

Other products and corporate names are trademarks or registered trademarks of their respective companies and are used only for explanation and to the owner's benefit; without intent to infringe.

© Copyright 2010 Don Johnston Incorporated. All Rights Reserved. Published by Don Johnston Incorporated. Printed in the United States of America.

Technical Support: support@donjohnston.com

Sales and Customer Service: 800.999.4660 USA/CANADA 847.740.0749

Web/Email: www.donjohnston.com info@donjohnston.com